




MEDICAL MONOFILAMENT MANUFACTURING

**Sensory Testing Monofilament 5.07/10g filament**

Pack of 20 or Pack of 40 Monofilaments

10gm Monofilaments are attached to an easy-to-use paperboard handle. Used to screen for loss of protective sensation (LOPS) and neuropathy. Featuring ADA recommended test sites on back of handle.

**\*Please select front of monofilament design before submitting order.**

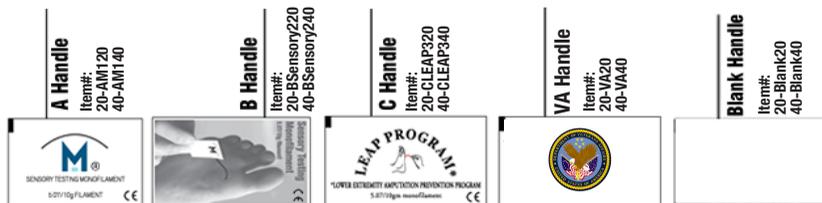

Monofilament pricing is the same in the 20 and 40 pack option, the only difference is the packaging. quantity

20 Pack: Item numbers ending in 20



40 Pack: Item numbers ending in 40



**Instructions For The Single-Use Monofilament**


1. Hold the filament by the paper handle.
2. Use a smooth motion to touch the filament to the skin on your foot for 1-2 seconds. Touch alongside of, and NOT directly on, an ulcer, callus, or scar. Push to make the filament bend.
3. Touch the filament in all sites circled on the drawing on back of monofilament.
4. Place a ⊙ in the circle if you can feel the filament at the site and a ○ if you cannot feel the filament.

Monofilaments are a screening device used to assess loss of protective sensation.  
Store at room temperature  
Medical Monofilament Plymouth, MA 02360  
Expiration Date:

Back of packaging

Back of blank Monofilaments

Back of all Monofilaments

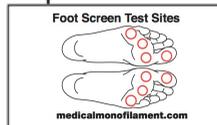

**20 PACK**

1-124 packs	13.00
125-249 packs	12.50
250-499 packs	12.00
500-1249 packs	11.50
1250-2499 packs	11.00
2500-4999 packs	10.50
5000+ packs	10.00

**40 PACK**

1-62 packs	26.00
63-124 packs	25.00
125-249 packs	24.00
250-624 packs	23.00
625-1249 packs	22.00
1250-2499 packs	21.00
2500+ packs	20.00



**M3 Monogripper**

Make the paper-handle Monofilament easy to hold and easy to use with our Monogripper. The Monogripper is a handheld plastic tool that firmly clasps a monofilament, allowing for an easier grip and simpler testing. The Monogripper can function as a permanent handle, even though our Monofilaments are disposable.

**\$9.99 ea**  
1000+=\$9.00 ea



MEDICAL MONOFILAMENT MANUFACTURING

**Sensory Testing Monofilament 5.07/10g filament**

Monofilaments Attached to Individual Instruction Cards - 25 per pack

ADA recommended test sites on the back of the instruction cards include instructions on appropriate use of the monofilament. Card is approximately business card size (3.5" x 2"). A Monofilament Handle is adhered to the inside of the instruction card. 25 per pack.

**\*Please select front of instruction card and monofilament design before submitting order.**



**25 PACK**

1-99 packs	24.00
100-199 packs	23.00
200-399 packs	22.00
400-999 packs	21.00
1000-1999 packs	20.00
2000-3999 packs	19.00
4000+ packs	18.00

**Instructions For The Single-Use Monofilament**

1. Hold the filament by the paper handle.
2. Use a smooth motion to touch the filament to the skin on your foot for 1-2 seconds. Touch alongside of, and NOT directly on, an ulcer, callus, or scar. Push to make the filament bend.
3. Touch the filament in all sites circled on the drawing on back of monofilament.
4. Place a ⊕ in the circle if you can feel the filament at that site and a ⊖ if you cannot feel the filament.

Monofilaments are a screening device used to assess loss of protective sensation.

Store at room temperature  
Medical Monofilament Plymouth, MA 02360

Expiration Date:

**Back of packaging**

**Inside of all Instruction Cards**



**A Handle Monofilament adhered to inside of Instruction Card**



**Back of all Monofilaments**

